



1. Alien Ear Monitors require a full impression and must include the crus of the helix, antitragus and tragus (just passing the second bend of the ear canal)
2. Foam or cotton ear dams are fine
3. Impressions should be open mouth (using a bite block) if the client is singing or makes faces on stage while using the monitors/molds please have them mimic these facial expressions while the impression material is in the ears.
4. Please put plenty of backing impression material on the impressions (i.e Frankenstein bolts!!!)

Please contact us with any questions Alien Ears
Phone: (727) 346-6483

All orders must be accompanied by an Alien Ears signed invoice/signature page that has been sent to the clients email address.